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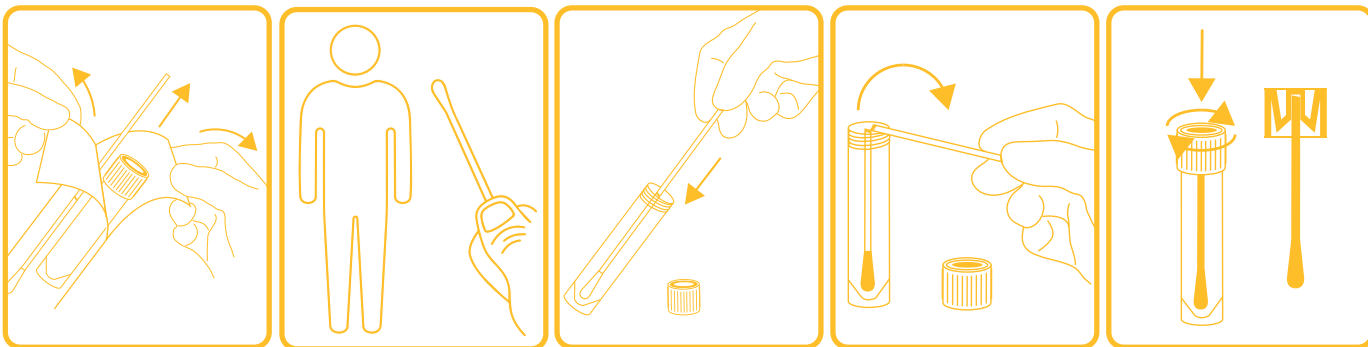
HOW TO USE / INDICAZIONI D'USO

APPLICATOR  
TAMPONE

Swabs with Flocked, Foam, Polyester or Rayon tip and plastic shaft. They allow a suitable clinical specimen collection and subsequent release into liquid transport medium. Plastic shaft with "break-point" to allow the swab breaking into the test tube and the successive collection by the screw "capture" cap after the tube closing. It allows to dispose of the swab together with the cap in a safer way.

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Synthetic fiber Flocked swab that allows an excellent clinical sample collection and the following immediate release into the liquid transport medium. An ideal choice with considerable absorption and elution of specimens. High purity characteristics makes it an ideal swab for various rapid diagnostic tests. It has a plastic shaft with "break-point".

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Available in three versions on sampling site base:

- **Standard tip:** swabs standard dimensions, for buccal, nasal, throat, vagina, groin, rectal, faeces, cutaneous and wound sampling;
- **Fine tip:** swabs thin dimensions, for nasopharyngeal, oropharyngeal, urethral, nasal, ocular and paediatric sampling;
- **Paediatric tip:** swabs small dimensions for paediatric and neonatal sampling.

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FOAM SWABS  
TAMPONI IN FOAM

Special polyurethane foam, medical grade, swab tip that allows a very good clinical sample collection and the following immediate release into the liquid transport medium. The soft foam bud is more comfortable for patients, and has significant advantages for both conventional and molecular methods. It has a plastic shaft with "break-point".

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Available in two versions on sampling site base:

- **Standard tip:** swabs standard dimensions, for buccal, nasal, throat, vagina, groin, rectal, faeces, cutaneous and wound sampling;
- **Fine tip:** swabs small dimensions, for paediatric, urethral, urogenital, ocular, nasal and nasopharyngeal sampling.

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POLYESTER OR RAYON SWABS  
TAMPONI IN POLIESTERE O VISCOSA

Swabs with Polyester or Rayon tip and plastic shaft. They allow an adequate specimen collection and subsequent release into liquid transport medium. Plastic shaft with "break-point".

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Available:

- **Standard tip:** standard dimensions, for buccal, nasal, throat, vagina, groin, rectal, faeces, cutaneous and wound sampling.

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CliniSwab<sup>VIRUS</sup> is a room temperature stable viral transport medium suitable for the transportation and preservation, from the collection site to the testing laboratory, of clinical samples containing Viruses (Sars-CoV-2, Sars-CoV-1, MERS-CoV, Influenza A (H1N1), Adenovirus, Cytomegalovirus, Herpes Simplex Virus, Parainfluenza, Respiratory Syncytial Virus, Varicella Zoster Virus, HPV - Human Papilloma Virus, etc.), Chlamydia, Mycoplasma and Ureaplasma specimens and can be used in the laboratory to perform culture. It consists of a balanced buffer solution to maintain neutral pH, anti-microbial agents and a preservative. Organisms viability maintained at room temperature and on ice. Suitable also for long term freeze storage. This product is not intended to be used for the collection and transport of general bacterial and fungal specimens. Test tubes with red screw cap test tube containing 1 ml or 3 ml of liquid transport medium for viruses, pre-labelled to allow a correct sample identification. CliniSwab<sup>VIRUS</sup> are sterile (SAL 10<sup>-6</sup>), pyrogen free, non cytotoxic.

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